

Product name	Manufacturer	Composition	Compatible with FibroScan® probe ultrasound localization probe	Manufacturer claims effectiveness against SARS-CoV-2
Aniospray Quick	ANIOS	Ethanol (50 % ≤ x ≤ 100 %), Propan-2-ol (1 ≤ x < 2,5 %), N,N-Didecyl-N-Methyl-Poly (Oxyethyl), Ammonium Propionate (0,1 ≤ x < 2,5 %)	✓✓	✓
Cleanisept Wipes	THX MEDICAL	100 g contains: 0,25 g didecyldimethyl ammonium chloride, 0,50 g Quaternary ammonium compounds, benzyl-C12-16-alkyldimethyl-, chlorides	✓✓	✓
Clinell Sporicidal Wipes	GAMA HEALTHCARE	Sodium Percarbonate (40 - 50 wt%), Citric Acid (≤ 15 wt%), Tetra acetyl ethylene diamine (10 - 35 %)	✓	✓
Clinell Universal Sanitising Wipes	GAMA HEALTHCARE	Benzalkonium chloride (≤0,5 %), Didecyl dimethyl ammonium chloride (≤0,5 %), Poly(hexamethylenebiguanide) (PHMB) (≤0,10 %)	✓✓	✓
LYSOL® ADVANCED™ Disinfecting Wipes	LYSOL	Ethanol (1-5 %), Quaternary ammonium compounds, benzyl-C12-16-alkyldimethyl, chlorides (0,1-1 %)	✓✓	✓
Meliseptol Foam Pure	B BRAUN	Propan-1-ol (< 20 %), Didecyldimethylammonium chloride (< 0,25 %)	✓✓	
Meliseptol Wipes Sensitive	B BRAUN	Propan-1-ol (< 20 %), Didecyldimethylammonium chloride (< 0,25 %)	✓✓	
Mikrobac Tissues	BENER SAFETY	Benzyl-C12-18-alkyldimethylammonium chlorides (0,25 ≤ x < 1 %w/w) Didecyldimethylammonium chloride (0,25 ≤ x < 1 %w/w)	✓✓	
Mikrozid Sensitive Wipes	SCHUELKE	Didecyldimethylammonium chloride (0,1 ≤ x < 0,25 %), Quaternary ammonium compounds, benzylC12-16-alkyldimethyl, chlorides (0,1 ≤ x < 0,25 %), Quaternary ammonium compounds, C12-14- alkyl [(ethylphenyl)methyl] dimethyl, chlorides (0,1 ≤ x < 0,25 %)	✓✓	✓
PDI Sani-Cloth AF3	PDI HEALTHCARE	Benzyl-C12-18-alkyldimethyl ammonium chlorides 0,14 %, Quaternary ammonium compounds, C12-18-alkyl [(ethylphenyl)methyl]dimethyl, chlorides 0,14 %	✓✓ <i>listed on EPA website (USA)</i>	✓
Protex ULTRA	PARKER LABS	Decyldimethyloctylammonium chloride 0,0909 %, Dimethyldioctylammonium chloride 0,0364 %, Didecyldimethylammonium chloride 0,0545 %, Quaternary ammonium compounds, benzyl-C12-16-alkyldimethyl, chlorides 0,1212 %	✓✓	✓
SONO Ultrasound Wipes	SONO HEALTHCARE	Decyldimethyloctylammonium chloride (0,0909 %), Dimethyldioctylammonium chloride (0,0364 %), Didecyldimethylammonium chloride (0,0545 %), Quaternary ammonium compounds, Benzyl-C12-16-alkyldimethyl, chlorides (0,1212 %)	✓✓ <i>listed on EPA website (USA)</i>	✓
Surfa'Safe Preium	ANIOS	Didecyldimethylammonium chloride (0 ≤ x ≤ 2,5 %), Amines, n-C12-14-alkyltrimethylenedi- (0 ≤ x ≤ 2,5 %)	✓✓	
Tristel Trio Wipe System Pre-Clean Wipe Sporicidal Wipe & Activator Foam Rinse Wipe	TRISTEL	Pre-Clean Wipe: No hazardous ingredient specified. Tristel Sporicidal Wipe – Activated Wipe : Chlorine dioxide (< 1%) Rinse Wipe : No hazardous ingredients specified.	✓	✓
Viraclean	WHITELEY	Ethanol (< 10 %)	✓ <i>listed by Therapeutic Goods Administration</i>	✓
V-Wipes	WHITELEY	Ethanol (< 10 %)	✓	
Wip'Anios Excel	ANIOS	Chlorure de didecyldiméthylammonium (2,5 % > c ≥ 0 %), Amines, N-C12-14-alkyltriméthylènedi (2,5 % > c ≥ 0 %)	✓	✓